

Orthoplan™ for VetPACS User Manual

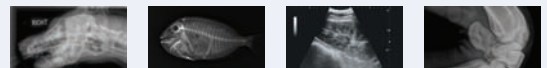


Table of Contents

Chapter 1: Before Installing	1
Chapter 2: Installing Orthoplan™	2
Chapter 3: Entering the License Key	3
Chapter 4: Updating Surgical Templates	4
Chapter 5: Launching the CD Training Videos	5
Chapter 6: Launching Orthoplan™ from VetPACS.....	6
Technical Support	7


Chapter 1: Before Installing

Before proceeding with the installation, first verify that your computer is able to connect to the necessary internet components to register Orthoplan™.

 **Orthoplan™ software must be installed on a computer that has Vetpacs on it.**

To Begin

Access <http://ortho.securelm.net/LicenseIssueWs/IssueService.asmx>.

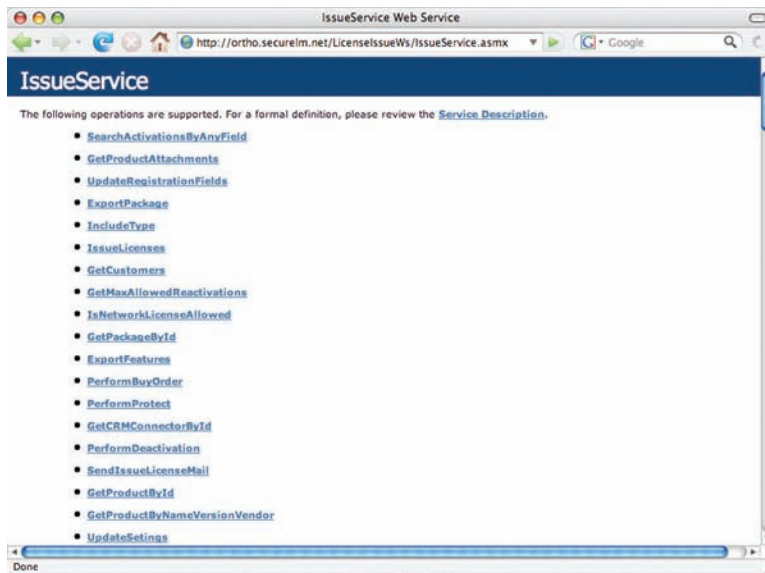

If you see a web page similar to the screen above, you will be able to connect to the Orthoplan™ server correctly. Please proceed to Chapter 2.

If you are unable to access:

<http://ortho.securelm.net/LicenseIssueWs/IssueService.asmx>



If you cannot view the web page screen shown above, you are either not connected to the internet or a firewall / proxy is blocking the connection. Please contact your system administrator to open this connection.

Chapter 2: Installing Orthoplan™

Install VetPACS Orthoplan™ software using the included CD-Rom. Since you have completed the internet access check detailed in Chapter 1, you may begin.



*Internet access is also necessary to register Orthoplan™
(shown in Chapter 3.)*

1. Access using your internet web browser. You should be able to view a list of services on the web page. (If you do receive any errors you or do not see any list, you are either *not* connected to the internet or there is a firewall/proxy blocking access.)
2. Place the “VetPACS Orthoplan™: Training & Setup” CD in your CD drive.
3. Browse to the CD by navigating to the Start Menu and selecting My Computer and choosing your CD-Rom drive.
4. Double click on the “**VetPACSOrthoplan™Setup.exe**” file to launch to Installation Wizard and begin set-up.
5. At the “Welcome to the InstallShield Wizard” screen click the Next button.
6. At the “License Agreement” window, click the “**I accept the terms in the license agreement**” button and click Next.
7. At the “Customer Information” window, you may customize the User Name and Organization. Ensure “Anyone who uses this computer (all users)” is chosen under the “Install this application for:” setting and click **Next**.
8. At the “**Ready to Install the Program**” screen, click the **Install** button. VetPACS Orthoplan™ will begin to install (please allow a few minutes for the software to install).
9. When you view the InstallShield Wizard Completed screen, click the **Finish** button.

Chapter 3: Entering the License Key

Validates your software using a unique identifier key.

1. Double click the VetPACS Orthoplan™ icon now located on your desktop to launch the Orthoplan™ software.



2. When prompted, enter the Activation Key* that is printed on the front of the CD-Rom (also found in Orthoplan™ documentation). Enter the key exactly as it appears (case sensitive and dashes must be included).

EXAMPLE LICENSE KEY: 1-2314-56A7-8AAA-912A

3. Click the **Online Activation** (green circle) button to register VetPACS Orthoplan™.
4. Enter the required registration information and press OK. Orthoplan™ will now launch.




Each license key is good for one computer only.




Chapter 4: Updating Surgical Templates

Updates the available surgical templates needed for some procedures.


1. Start a VetPACS Orthoplan™ session by double clicking the Orthoplan™ icon located on your desktop. Click the **Download Templates** button on the left hand side of the screen.

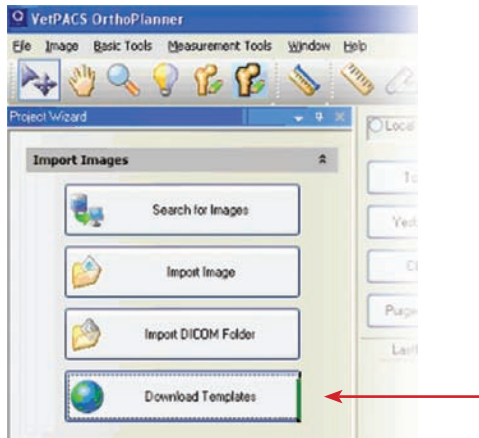

2. Select the Templates you would like to download by left clicking on the boxes with the cursor.

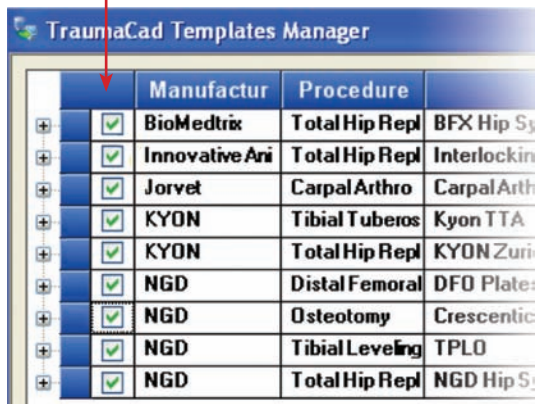

3. When finished, select **Download Selected Templates** from the bottom of the window.

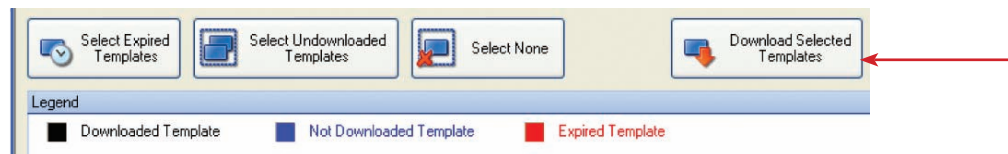

4. Once the templates have downloaded, close the TraumaCad Template Manger and exit out of Orthoplan™.

Chapter 5:

Launching the CD Training Videos

Launches the four training videos included on the CD-Rom.

1. Browse to your CD-Rom drive. Open the folder entitled **Orthoplan™ Training Software**.
2. Open the file `autorun.bat`.
3. Your internet browser will open and allow you to select four different training videos. The following training videos are included:
 1. Customization
 2. Launch & Calibrate
 3. Tibial Plateau Leveling Osteotomy (TPLO)
 4. Distal Femoral Osteotomy (DFO)
4. When a video is finished, press the back button on your browser to return you to the main menu and choose another video.

Chapter 6:

Launching Orthoplan™ from VetPACS

Seamlessly launch Orthoplan™ from VetPACS software while reviewing an exam.

1. Open your VetPACS software.
2. Choose a patient and an exam. Then select the **Exam Review** button.
3. Once in the Exam Review screen, select the **Orthoplan™** button to launch the Orthoplan™ software.



Technical Support

Shipping Address

Sound-Eklin™
5817 Dryden Place, Suite 101
Carlsbad, California 92008
USA

Technical Support

Office hours: Weekdays 8:00 A.M. – 5:00 P.M. Pacific Time
Emergency 24-hour support is available.
Telephone: 800-268-5354 option 3

Website

www.soundeklin.com



a VCA ANTECH company

Sound-Eklin™ Technical Support

5817 Dryden Place, Suite 101
Carlsbad, California 92008 USA

toll free: 800.268.5354 option 3
phone: 760.918.9626
fax: 760.918.9620
international: +1.760.918.9626

www.soundeklin.com

© 2010 All Rights Reserved
Sound-Eklin™ — a VCA ANTECH company